

Zitzi
PENGY



Anatomic
SITT™

Invented for a better life ♥

**Updated
product!**



Zitzi Pengy Pro


for a well positioned seating



Zitzi Pengy Pro – a stroller with well positioned seating


Step 1. Pelvic and leg positioning

The combination of correctly adjusted seat part, generous seat cushion, effective hip belt and good support for sacrum provides the conditions for reduced shear which facilitates activity and postural control. The seat section is infinitely adjustable in seat depth and in abduction/adduction, which enables optimal individual sitting position and at the same time allows the patient to grow with the seating system.

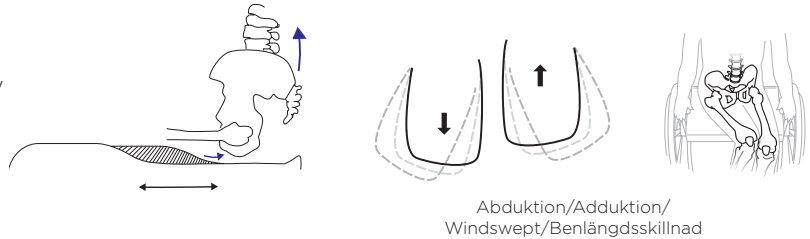


Step 2. Hip belt

A properly adjusted positioning belt gives the right condition in order to positioning pelvis and reduce the risk of shear. We have a big range of positioning belts that includes several models and variants, such as different materials, models and sizes.

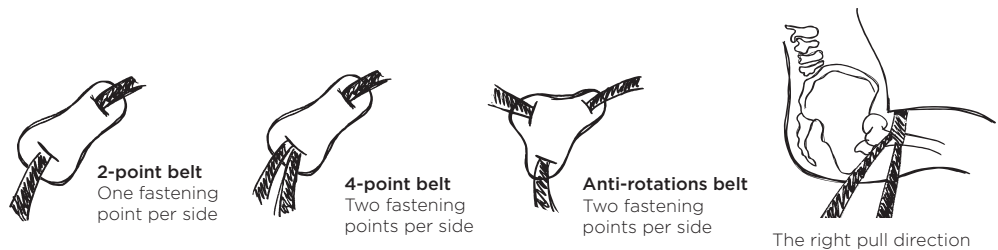


Step 3. Three-point positioning

Adjustable lateral supports together with eg. hip support creates a good condition for positioning of the torso, which reduces the risk of misalignment. Good postural control leads to better head control and increases the opportunities for activity and alertness. We have the opportunity to both correct and maintain the position of the chest based on the conditions we are facing.

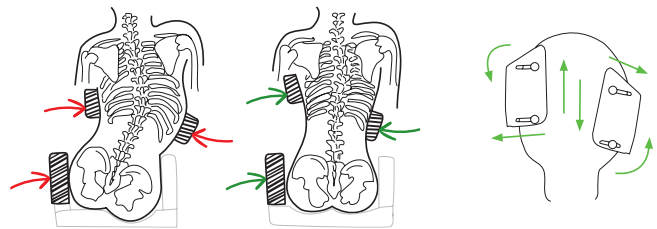

Harnesses


Together with the hip belt provides the harness a good support for positioning the trunk and provide conditions for a better head control. We offer different positioning harnesses;

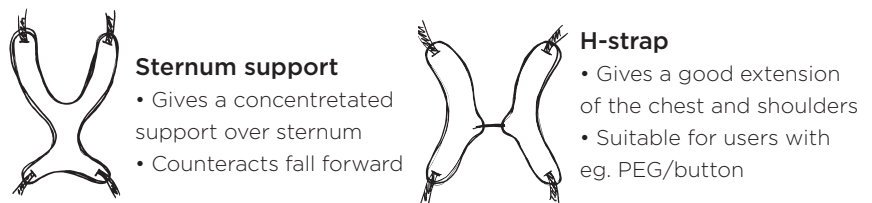

- Sternum support**
- Gives a concentrated support over sternum
 - Counteracts fall forward

- H-strap**
- Gives a good extension of the chest and shoulders
 - Suitable for users with eg. PEG/button

Step 4. Flex operation / hip angle adjustment

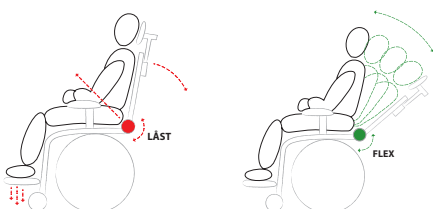

Dynamic support provides freedom of movement, variation and can provide a calming effect in extensional spasticity. The seating systems are equipped with a flex joint that makes the backrest compliant. The back is adjustable in the hip angle which breaks the spasticity and the flex joint captures the force and retains positioning in the sitting position.

Zitzi

Pengy Pro seat

The Pengyseat is in a way similar to Delfi Pro and special developed for the Pengy wagon. The Pengyseat has the same function as Delfi Pro such as adjustable trunksupport, seat adjustments, flex-function, along with other good features.

Accessories - included:

Armrest
Hipbelt
Lining

Sun cover (Extra choice)

Oppertunity with a sun-cover to protect the child against rain and sun.



Armrest

Stable armrest with adjustment possibilities; height, width and depth.

Adjustable seat part

The seat part är adjustable in seat depth and in abduction/adduction. The system can grow with the child.



Fabric color:

Black, blue, orange

Seatdepth

Size 1: 19-28 cm
Size 2: 28-38 cm

Max seatwidth


Size 1: 31 cm
Size 2: 35 cm

Coolmax®

Lining with Coolmax fabric that has a cooling effect and gives a comfortable temperature.

Hipbelts and harnesses

The harnesses together with the hipbelt provides a good support for positioning the trunk and provides for a better head control. We offer different positioning harnesses: Sternum support and H-strap are the most common.

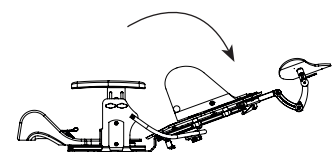

Zitzi Pengy Pro

Possibility to 75° lying position

PENGY WAGON

SIZE.....	1
WHEEL.....	front 8", back 12"
LOAD WEIGHT.....	47 kg
TILT.....	-35° to +35°
SEAT HEIGHT.....	54 cm
COLOR.....	Black
WEIGHT.....	12 kg
CRASHTESTED.....	Approved to 43 kg, for forward sitting according to ISO 7176-19



Easy to drive

Thanks to the right balance and lightly rolled wheels, the Pengy Wagon is great both for a city walk or a walk in the forest. The wagon can be equipped with fixed 12" casters in front to facilitate driving during winter season or in terrain.

Easily folded

The Pengy Sulky can easily be folded together to facilitate transportation by car or for storage.

The wheels rolls easily with a good balance and they are equipped with direction lock, quick release and suspension as standard.

Reversible seat

The seat part is reversible and removable. Simply turn the seat so that the child can ride either facing forward or backwards.

Suspension chassi

Pengy wagon has a integrated suspension which makes it easy to roll on uneven surfaces.

Crashtested

Crashtested and approved according to ISO 7176-19 to travel safely during transport (seated forward).



Zitzi

Pengy wagon

Freedom through combination



PENGY PRO

DELFI PRO

ANATOMIC SITS & SOFT SITS

CARSEAT PRO



Accessories Pengy Pro seat



385215 Black lining



385216 Blue lining



385217 Orange lining




385230 Table



385236 Headrest size 1 black



385237 Headrest size 1 long black

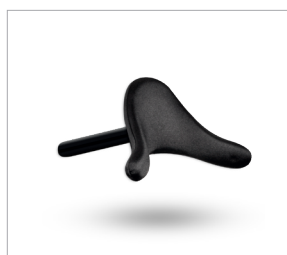

77011 Pro Mini size S
77012 Pro Mini size M



77004 Pro Plush Adjust. 6-14"



Bracelet w. pad and elastic band. Size S/M/L



Ankle holder 3-p with buckle or velcro. Size XS-XL




285058 Hip belt Soft size 2-3



374001 Sternum support size XS
374002 Sternum support size S
374003 Sternum support size M



374015 H-strap size XS
374016 H-strap size S
374017 H-strap size M

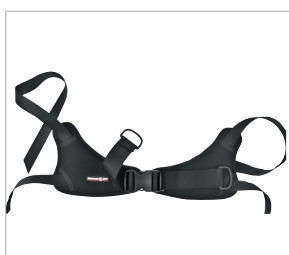


374060 Anti-rotation belt size XS
374061 Anti-rotation belt size S



990070 Suncover - Zitzi (need to be complemented with 990147 bracket suncover cpl.)



385229 Rain protection

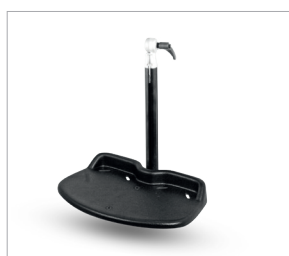

285070 Foot rest size 0, cpl.



281110 Calf support size 1 - Zitzi



281111 Padded calf support size 1



385244 Cross bar size 1

Accessories Pengy wagon



285072 Footrest size 2 - Zitzi



281110 Calf support size 1 - Zitzi



385222 Footrest Pengy std.



385223 Calf support size 1



990042 Footbox size 2 (whole foot plate) (fits to foot rest 285072)




385241 Adapter plate Carseat Pro - Zitzi Pengy



385242 Adapter plate Universal - Zitzi Pengy



385243 Adapter plate Pengy - Zitzi frames

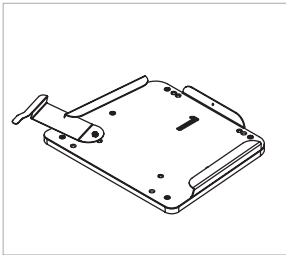

26531/26532/26533 Adapter plate reinforced size 1/2/3



285171/285172/285173 Mounted system size 1/2/3



153047 Adapter system Pro (cpl. 153048+153049)
153048 Adapter plate Pro (mounted on under frame)
153049 Adapter plate Pro (mounted on under frame)



385220 12" wheel front (replaces std.) - Zitzi Pengy




385227 Basket



385290 Footrest bracket



385232 Transport bracket

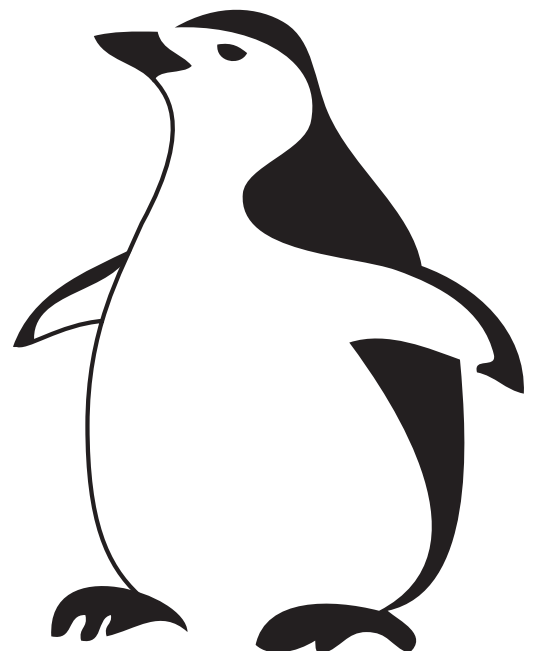



Anatomic SITT AB

Box 6137, SE-600 06 Norrköping

Telefon +46 11-16 18 00

Email info@anatomicsitt.com

Web www.anatomicsitt.com